PREMIOFORM

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ENGLISH

Intended use:

The PREMIOFORM nail correction system is used to treat deformed and ingrown fingernails and toenails in order to promote physiological nail growth, relieve pain, and straighten the nail shape.

The effect is based on the slight lifting of the nail edges by a mechanically generated tensile force of the ACTIVATION BRACE and the subsequent stabilization by means of a light-cured plastic compound consisting of the components BASE and FLUIDGEL.

Indications:

Unguis convolutus (curled nail), unguis incarnatus (ingrown nail) stages 1–3, excluding the contraindications listed below, and for recurrence prevention after surgical treatment or completed orthonyxia treatment.

Duration of use:

The duration of use depends on the initial findings and individual nail growth. Follow-up checks should be determined based on the patient's medical history, daily if necessary. The product must be removed after 6 months at the latest. If further treatment is required, use the PREMIOFORM nail correction system again.

SAFETY INSTRUCTIONS

Contraindications:

The PREMIOFORM nail correction system may only be used in the following cases after careful consideration of the risk-benefit ratio and, if necessary, in consultation with a physician: Onychomycosis, onycholysis, growth arrest, tumorous changes, subungual granulation, onychorrhexis, PAD (peripheral arterial disease) (in individual cases), polyneuropathy/angiopathy (in individual cases), psoriasis (in individual cases), allergic reactions to cyanoacrylate ethyl ester.

User group:

The PREMIOFORM nail correction system may only be used by specialists with the appropriate professional qualifications. Users must be familiar with the necessary hygiene standards and have experience in recognizing and assessing treatment risks. Liability for the consequences of improper use is excluded.

General patient and user protection:

- Before each treatment, clean and disinfect the nail to be treated and the surrounding skin.
- Wear sterile disposable gloves and, for your own protection, also use a face mask and safety goggles.
- Use the braces exclusively as described in these instructions for use and with original accessories.

Risks:

Infections and hygiene

- A pre-damaged or infected nail plate and inadequate hygiene can lead to nail fungus or other infections.
- Hygiene standards must be strictly adhered to. Disposable application brushes may only be used on one patient to avoid cross-contamination, especially mycoses. They must then be disposed of properly immediately.

Handling OrthoGlue (cyanoacrylate adhesive)

- OrthoGlue bonds skin and eyelids within seconds and irritates the eyes, respiratory tract, and skin.
- Avoid contact with eyes, skin, wounds, and mucous membranes at all costs; do not inhale vapors.
- Wounds must be covered before application.
- In case of eye contact or ingestion, seek medical attention immediately.
- Do not use force to remove any skin adhesions; instead, soak them in warm soapy water for a long time or carefully remove them with acetone.

Handling PREMIOFORM BASE and FLUIDGEL

- Avoid skin and eye contact with PREMIOFORM BASE and FLUIDGEL, as irritation or allergic reactions may occur.
- In case of contact with skin, immediately wash the affected area thoroughly with soap and water.
- In case of contact with eyes, rinse immediately with plenty of water for at least 10 minutes and seek medical attention.

UV radiation

- Do not look directly into the light while using the PREMIOLUX light curing device, as UV radiation can damage the eyes.
- If you experience any discomfort in your eyes during irradiation, stop the treatment immediately and consult a doctor.

Vapors and grinding dust

- Only use the system in well-ventilated rooms.
- Avoid inhaling fumes and dust produced during sanding, as they can irritate the respiratory tract.

Special events:

If unusual symptoms, severe pain, inflammation, or other unusual occurrences arise, remove the product immediately, disinfect the affected toe, and apply a sterile bandage. Consult a doctor immediately.
Report any unusual occurrences to 3TO GmbH (+49 89 4520833-0 / info@3to-gmbh.de).

Preparing the working materials

- ⚠ Sterilize the cutting pliers before each brace treatment.
- ⚠ Check that the PREMIOLUX light therapy device is working properly before treatment - charge it if necessary.

Preparing the nail

- 1) Remove any loose parts from the nail surface. If necessary roughen the nail with a file or mill.
- 2) Clean the nail with an alcohol swab and allow the alcohol to dry thoroughly (at least 30 seconds).

Attaching the ACTIVATION BRACE

- 3) Select the size so that the ACTIVATION BRACE covers as much of the nail width as possible.

The expansion of the ACTIVATION BRACE on the nail (approx. 0.7 - 2.7 mm) must be taken into account. Therefore, select active elements that are shorter than the nail width by the expansion length.

The active element should be placed as close as possible to the lateral edges of the nail and as proximally as possible.

- 4) Clean and degrease the ACTIVATION BRACE thoroughly with a fresh alcohol swab and allow the alcohol to dry completely (approx. 30 seconds).

- 5) Unscrew the adhesive and lightly wipe the brush on both sides once each on the edge of the bottle. Apply a thin layer of OrthoGlue to the adhesive surface A of the pad.

Use the adhesive sparingly!

If you use too much adhesive, the brace will not stick to the nail and the adhesion between the nail and the gel will be impaired by excess adhesive.

Keep the rim and thread of the bottle clean. Clean them if necessary, e.g., with a alcohol swab.

6) Position adhesive surface A on the side of the nail that is more ingrown.

Press adhesive surface A onto the nail with your finger for approx. 20 seconds (do not move during this time!).

7) The adhesive must then be left to harden for 2 minutes without further contact.

Wait at least 2 minutes!

8) Gently wipe the OrthoGlue brush on both sides. Apply a thin layer of adhesive to adhesive surface B of the ACTIVATION BRACE. Make sure that no upward force is applied to the glued side A and that it is not pulled off.

9) The ACTIVATION BRACE is stretched by 0.7 to 2.7 mm (depending on the width of the brace, the thickness of the nail, and the findings). Pull the ACTIVATION BRACE parallel to the nail. Place the brace on the nail in the stretched state and press the adhesive surface B for approx. 30 seconds. Make sure that the active element is not moved after placement.

10) The adhesive must then be allowed to harden for 2 minutes without further contact.

Wait at least 2 minutes!

11) Place the pliers on the connections to the handle from above and pinch off the handle.

Applying the BASE

12) Apply a small drop of BASE to the recess in the ACTIVATION BRACE and spread it with the application brush over all gel chambers up to the edges. Excess BASE can be removed with the applicationbrush.

Do not apply too much, just wet the nail surface and brace!

13) Activate the PREMIOLUX curing device by moving the slide switch forward and holding down the power button until the 60-second indicator flashes. Cure the BASE with the PREMIOLUX curing device for 60 seconds.

Do not remove the dispersion layer afterwards!

14) Clean the application brush with the alcohol swab.

Applying the FLUIDGEL

15) Place a drop of FLUIDGEL into the recess of the ACTIVATION BRACE and spread it over all gel chambers with the application brush until the ACTIVATION BRACE is completely filled to the top edge.

16) Cure the FLUIDGEL completely with the PREMIOLUX lamp for 60 seconds.

17) Wipe off the dispersion layer with the alcohol swab.

18) Dispose of the disposable application brush.

19) If the gel layer is too thick, it can be filed down with a file/drill.

Removing the nail correction system

To remove the nail correction system, lift the edges of the brace with the pliers. The brace can then be pulled off. Any adhesive and gel residue can be removed from the nail with a file or drill.

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3TO GmbH – Ihr Spezialist für Nagelspangen