**podofix**®

Gebrauchsanweisung  
Instructions for use  
Gebruiksaanwijzing  
Manuel d'instructions  
Istruzioni per l'uso  
Instrucciones de uso

DE

EN

NL

FR

IT

ES

01.08.2022

p\_DE\_EN\_NL\_FR\_IT\_ES\_2022

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# Safety Instructions

### **Purpose:**

The therapy with nail clips from 3TO GmbH serves to restore the natural growth of the nail. With the braces, pain can be relieved or the growth of ingrown and/or deformed nails can be corrected by lifting the ingrown nail. The brace is a disposable product and is delivered non-sterile. Nail correction braces are Class I non-invasive medical devices.

### **Indications:**

Unguis incarnatus (ingrown nail), Unguis convolutus (rolled nail), Onychophosis (excessive hornification in the nail fold), Clavi in the nail fold (corn), after operations to prevent recurrence.

### **Contraindications:**

The braces may only be used for the following clinical pictures after careful consideration of the benefit-risk ratio and, if necessary, in consultation with a doctor Onychomycosis, onycholysis, growth arrest, tumorous changes, subungual granulation, onychorrhexis, pAVK - peripheral arterial occlusive disease - (in individual cases), polyneuropathy / angiopathy (in individual cases), psoriasis (in individual cases), allergic reactions to ethyl cyanoacrylate.

### **Group of users:**

To ensure the protection of the patient, the nail correction braces from 3TO GmbH may only be used by medically trained specialists (e.g. podiatrists or doctors). The medical expertise to assess the risks of the therapy as well as the ability to handle instruments are basic requirements for the application of nail correction braces. The liability for consequences of improper use shall be excluded herewith.

### **Duration of the application:**

The total duration of the therapy depends on the initial findings and the individual growth of the nail. The frequency of follow-up examinations with replacement of the tamponade fleece must be determined individually according to the medical history. If necessary, a daily check must be carried out. The braces usually have to be changed every 4 - 8 weeks.

### **Patient and user protection:**

Clean and disinfect the nail to be treated.

To protect the patient and user, the therapist must wear sterile, commercially available disposable gloves. For the therapist's own protection, he or she should wear a face mask and protective goggles during each treatment.

Only use the braces as described in this manual and with original accessories.

## **Risks:**

Risk of skin irritation, injury, inflammation and pain in the nail environment: The correct fitting and insertion of nail correction braces is an important prerequisite to avoid skin irritation, skin injury, pain, cornification and inflammation.

Risk of nail separation (onycholysis):

The correct adjustment of the pulling and levering forces of nail correction braces are important prerequisites to prevent the nail from loosening.

Risk of infection:

An infection of the nail plate, a pre-damage of the nail or a lack of hygiene can lead to a nail mycosis. Patient education should include ongoing hygiene measures to prevent infection.

Danger from flying parts:

When pinching off the wire, care must be taken to ensure that no parts fly around. Make sure that your face, the patient's face and any third persons are protected (e.g. by protective goggles). Place your fingers on the ends of the wire for protection when pinching off.

Danger from sticking:

The OrthoGlue adhesive bonds skin and eyelids together within seconds. Contains cyanoacrylate. Irritating to eyes, respiratory system and skin. Avoid contact with eyes and skin. In case of contact with eyes or internal Organs, please see your doctor immediately. Do not inhale vapours. May not come into contact with wounds - Cover wounds with oil or grease before use. Do not remove any skin adhesions by force but soak them in warm soapy water for a longer period of time or carefully loosen them with acetone.

Danger from UV radiation:

Under no circumstances look directly into the UV radiation of the micro lamp. It damages the eye.




The reactions usually occur quickly and are quickly noticed by the patient and user. Countermeasures must be taken immediately.

## **Special incidents:**

Should any special incidents occur when using products of 3TO GmbH, remove the brace immediately from the patient. The affected toe must be disinfected and covered with sterile dressing. Consult a doctor immediately. Please contact 3TO GmbH immediately by phone +49 89 4520833-0 (outside opening hours by e-mail to [info@3to-gmbh.de](mailto:info@3to-gmbh.de)). You will then be sent a „Special incidents“ form. This is to be filled in and sent to 3TO GmbH as soon as possible.

## **Patient information:**

Inform your patient about the causes and prevention possibilities of ingrown nails.

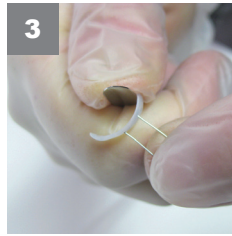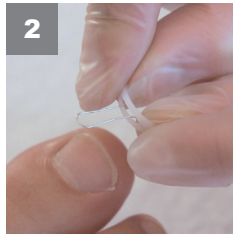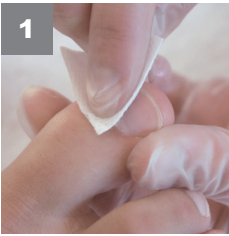- 1 Remove non-adhesive parts from the nail surface. **Smooth** extremely rough nails or roughen extremely smooth nails very lightly with a file or milling tool.

**Clean and degrease** the nail thoroughly with the alcohol swab and let it dry (about 30 seconds).

Health risk: danger of infection - see safety instructions


- 2 Choose the **right size**. The brace should be about as wide as the nail, by no means wider. It must not disturb or press in the nail fold. Bend the brace with your fingers a little more than the nail plate is curved, it will then fit better to the nail.

- 3 This is particularly important **at the ends** of the braces. You can bend them well over a thin edge (e.g. over the spatula or side cutter).

The braces are also cleaned and degreased with the alcohol swab and then allowed to dry (about 30 seconds).

Health risk: danger of infection - see safety instructions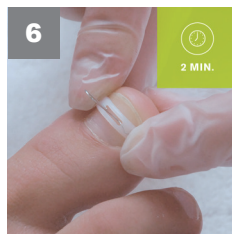

- 4 Open the adhesive and wipe the brush lightly from both sides of the bottle once at the bottle edge.

- 5** Wet the adhesive surface of the pad with **very little** adhesive. Apply the adhesive sparingly from the centre to the edges. Use the adhesive **sparingly**; if too much adhesive is used, the brace will not stick to the nail.

Keep the edge and the thread of the bottle clean. Clean them if necessary, e.g. with a swab.



Health risk: Danger due to sticking - see safety instructions

- 6** Position the brace on the **more ingrown side** of the nail. To do this, hold the brace with one hand on the wire hanger and place it smoothly on the nail.

Health risk: Danger due to sticking - see safety instructions

EN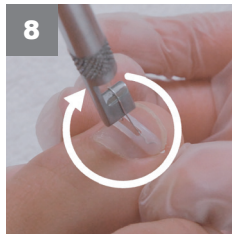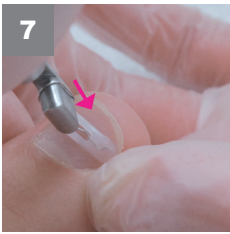



- 7** Grasp one side of the brace with the index finger or thumb of the free hand and press it against the nail without removing the finger. Now press the brace on the other side with the index finger or thumb of your second hand without releasing the first side. At the beginning of the pressing, pull the edges of the brace down as far as possible towards the nail fold, allowing the middle part of the brace to rest against the nail. Press the brace well against the nail, especially at the edges, **for about 15 seconds** and then remove your fingers.

Health risk: Danger due to sticking - see safety instructions

The adhesive needs **at least two minutes** to harden.

- 8** Insert the tensioning instrument into the wire loop. Turn it one complete turn. Make sure that the wire is twisted under the tensioner.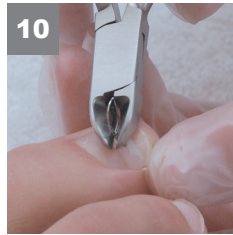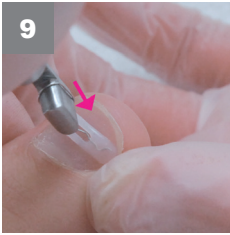

- 9** After one complete rotation, press the tensioning instrument down so that the wire spiral sinks into the recess of the brace.

Now the tension is built up by turning the tensioning instrument further. Turn the tensioning instrument slowly **for a maximum of another complete turn** until the desired tension is reached.


Health risk: Danger of nail detachment - see safety instructions

- 10** The wire is cut **in the wrap** with the side cutter. To do this, place the side cutter in a flat position on the brace and cut the wire loop with a cut parallel to the loop. One wire coil must remain. If wrappings of the wire still protrude over the plastic pad, you can press the wire loop into the recess with the tensioning instrument.

Health risk: Danger from flying parts - see safety instructions

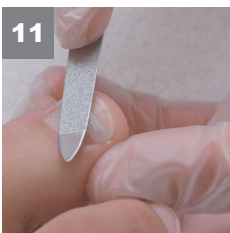- 11** Smooth the wire ends with a file.

- 12** For sealing, apply a drop of PediGel with the spatula and model it into the cavity. Activate the 3TO micro lamp and harden the gel by illumination for **+ 20 seconds**. Hold the micro lamp at the smallest possible distance (max. 5 mm) from the sealing point. A greasy layer will form on the surface of the gel during hardening, which can be easily removed with the alcohol swab.

Health risk: Hazard due to UV radiation - see safety instructions

EN

- 14** To remove the podofix brace, the edges of the brace are removed with the side cutter. The brace can then be pulled off.

Residual adhesive can be removed from the nail with a file or a milling tool.

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