DE

EN

NL

ES

COMBIped[®]

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Safety Instructions

Purpose:

The therapy with nail clips from 3TO GmbH serves to restore the natural growth of the nail. With the braces, pain can be relieved or the growth of ingrown and/or deformed nails can be corrected by lifting the ingrown nail. The brace is a disposable product and is delivered non-sterile. Nail correction braces are Class I non-invasive medical devices.

Indications:

Unguis incarnatus (ingrown nail), Unguis convolutus (rolled nail), Onychophosis (excessive hornification in the nail fold), Clavi in the nail fold (corn), after operations to prevent recurrence.

Contraindications:

The braces may only be used for the following clinical pictures after careful consideration of the benefit-risk ratio and, if necessary, in consultation with a doctor Onychomycosis, onycholysis, growth arrest, tumorous changes, subungual granulation, onychorrhexis, pAVK - peripheral arterial occlusive disease - (in individual cases), polyneuropathy / angiopathy (in individual cases), psoriasis (in individual cases), allergic reactions to nickel, allergic reactions to ethyl cyanoacrylate.

Group of users:

To ensure the protection of the patient, the nail correction braces from 3TO GmbH may only be used by medically trained specialists (e.g. podiatrists or doctors). The medical expertise to assess the risks of the therapy as well as the ability to handle instruments are basic requirements for the application of nail correction braces. The liability for consequences of improper use shall be excluded herewith.

Duration of the application:

The total duration of the therapy depends on the initial findings and the individual growth of the nail. The frequency of follow-up examinations with replacement of the tamponade fleece must be determined individually according to the medical history. If necessary, a daily check must be carried out. The braces usually have to be changed every 4 - 8 weeks.

In case of newly occurring changes such as mycosis/redness or infections and complaints in the area of the hooking point, the brace must be removed immediately and an antiseptic treatment must be carried out by a qualified therapist.

Patient and user protection:

Clean and disinfect the nail to be treated. To protect the patient and user, the therapist must wear sterile, commercially available disposable gloves. For the therapist's own protection, he or she should wear a face mask and protective goggles during each treatment. Only use the braces as described in this manual and with original accessories.

The braces must be disinfected in a bath or sterilized in a hot-air sterilizer or autoclave before use on the patient. It is imperative that you follow the manufacturer's instructions for correct use, in particular the exposure times prescribed by the manufacturer and the protocol.

Risks:

Risk of skin irritation, injury, inflammation and pain in the nail environment: The correct fitting and insertion of nail correction braces is an important prerequisite to avoid skin irritation, skin injury, pain, cornification and inflammation.

Risk of nail separation (onycholysis):

The correct adjustment of the pulling and levering forces of nail correction braces are important prerequisites to prevent the nail from loosening.

Risk of infection:

An infection of the nail plate, a pre-damage of the nail or a lack of hygiene can lead to a nail mycosis. Patient education should include ongoing hygiene measures to prevent infection.

Danger of stinging:

Particularly when adjusting the hooks of wire clips, care must be taken to ensure that no sharp edges are created (deburring). Incorrectly bent hooks or even improper use can lead to stab injuries. If there is a lack of disinfection in connection with a stab wound, there is also a risk of infection.

Danger from flying parts:

When pinching off the wire, care must be taken to ensure that no parts fly around. Make sure that your face, the patient's face and any third persons are protected (e.g. by protective goggles). Place your fingers on the ends of the wire for protection when pinching off.

Danger from sticking:

The OrthoGlue adhesive bonds skin and eyelids together within seconds. Contains cyanoacrylate. Irritating to eyes, respiratory system and skin. Avoid contact with eyes and skin. In case of contact with eyes or internal Organs, please see your doctor immediately. Do not inhale vapours. May not come into contact with wounds - Cover wounds with oil or grease before use. Do not remove any skin adhesions by force but soak them in warm soapy water for a longer period of time or carefully loosen them with acetone.

The reactions usually occur quickly and are quickly noticed by the patient and user. Countermeasures must be taken immediately.


Special incidents:

Should any special incidents occur when using products of 3TO GmbH, remove the brace immediately from the patient. The affected toe must be disinfected and covered with sterile dressing. Consult a doctor immediately. Please contact 3TO GmbH immediately by phone +49 89 4520833-0 (outside opening hours by e-mail to info@3to-gmbh.de). You will then be sent a „Special incidents“ form. This is to be filled in and sent to 3TO GmbH as soon as possible.


Patient information:

Inform your patient about the causes and prevention possibilities of ingrown nails.

Basic information on the use of the COMBIped brace: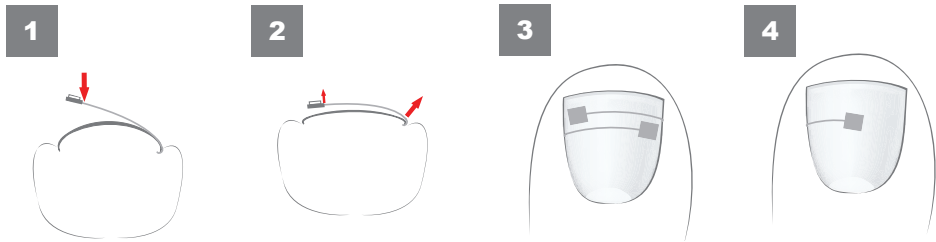


- 1** The more the nail is curved, the higher is the power of the brace. The shorter the brace, the higher the force on the hook.
- 2** The COMBIped brace generates a **high force on the wire side** and a lower force on the pad side.
- 3** If a **double-sided** effect is desired, the brace should be as long as possible and cover the entire width of the nail. Two COMBIped braces can also be placed one behind the other.
- 4** For a **one-sided** effect the brace is made shorter and the adhesive pad is glued in the middle of the nail. The effect is then concentrated on the wire side.

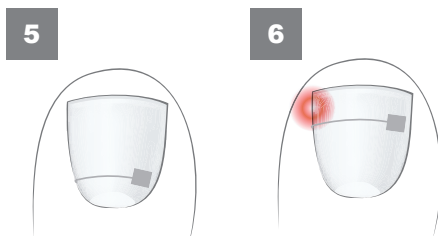- 5** To work out a strongly **rolled** nail, the COMBIped brace should be used in the **proximal area**.
- 6** For **acute pain** the COMBIped brace should be placed approx. 2 mm behind the pain area.

Preparation:

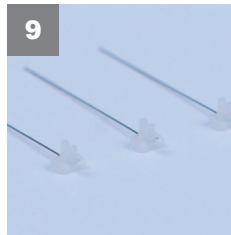7 Prepare nail and clean nail fold:

Remove non-adhesive parts from the nail surface. Smooth extremely rough nails or roughen extremely smooth nails very lightly with a file or burr. Before attaching the brace, clean the nail fold and create the conditions to insert the hook under the nail edge.

8 Disinfect the brace:

Disinfect the wire of the COMBIped brace by wiping it with an alcohol swab before bending the hook and before each insertion.

Health risk: danger of infection - see safety instructions

9 Selection of brace:

Choose the thickness of the brace according to the nail thickness (force requirement).

soft: wire thickness 0.25 mm - for soft, narrow and normal nails

medium: wire thickness 0.30 mm - for normal and thick nails



hard: wire thickness 0,35 mm - for extremely strong and very thick nails

The brace thickness „hard“ is only suitable for very thick, strong nails, so preferably use the thicknesses „soft“ and „medium“!

If too strong a brace is used, there is a risk that the nail will become detached. Therefore, make sure that the tension is moderate.

If the patient feels a strong tension, the brace must be removed and replaced by a weaker one.
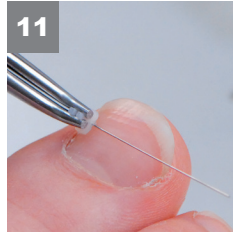
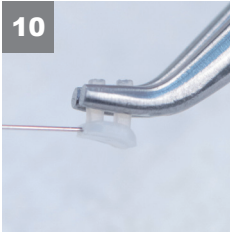


10 Grasping the bracelet:

Clamp the brace with the COMBIped holding pliers at the two pins (according to photo). Due to a small distance between pad and pliers the brace remains flexible in the clamping.


11 Determine the length of the brace:

Determine the length of the brace according to its intended use (one or both sides) and the width of the nail.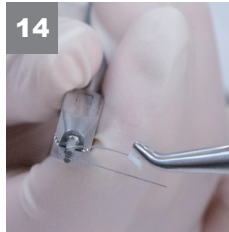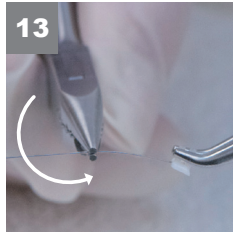
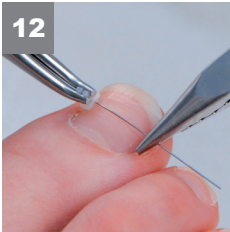


12 - 14 Bend the hook:

Use the combi bending pliers to bend a rough hook at the measured point. Always bend around the round mandrel of the combi bending pliers! Hold your finger against the wire from above so that it does not bend. Note: Do not apply pulling force lengthwise, otherwise the wire may be pulled out of the pad.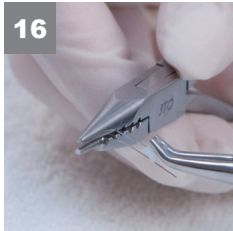

15 - 16 Depending on the desired hook size, one of the four holes is used. The drill holes - and thus the hooks - become smaller towards the tip of the pliers. Place the pre-bent hook **upright** in the selected hole and press the combi bending pliers together completely.

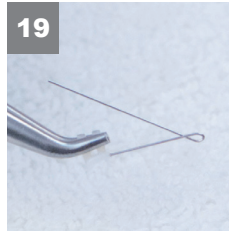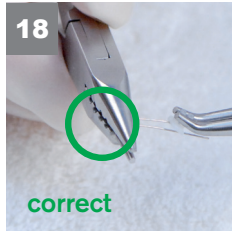
17 - 18 The hook **must not** stick out on the other side of the pliers. Alternatively, the hook can also be pressed together with the grooved, rear section of the pliers.

19 - 20 Cut off the hook briefly with the side cutter. The **flat side** of the side cutter points to the remaining hook.


Health risk: Danger from flying parts - see safety instructions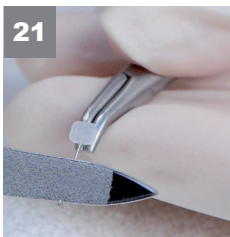

21 - 22 **Deburr** the hook with a file or a milling tool. Support the wire with a finger when doing so.

EN

Insert the brace: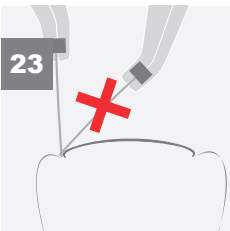

CORRECT ANGLE

This is the only way to turn the hook well under the nail edge.

WRONG ANGLE



The hook cannot be turned under the nail edge and injures the nail fold.

Health risk: Danger of stinging - see safety instructions

Important: In cases of inflammation with or without granulation tissue, the nail fold in the area around the brace legs must be tamponaded before insertion. Contact between the brace material and the inflamed tissue must be avoided.
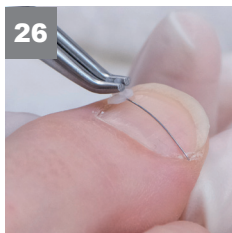
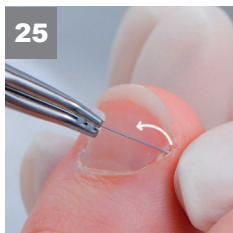


24 - 26 To **insert the brace**, insert the hook into the nail fold. The open side points to you. Turn the hook under the edge of the nail and then press the pad onto the nail with the holding pliers. Pull lightly with the pliers so that the hook does not pop out.

If you pull too hard, the wire can be pulled out of the pad and the brace can be destroyed.

If the hook jumps out, it is usually helpful to bend the wire slightly just before the hook.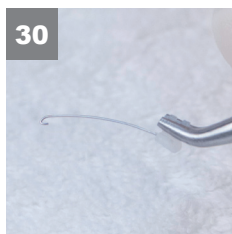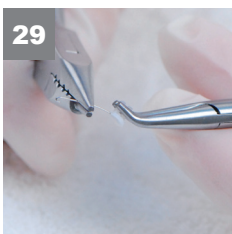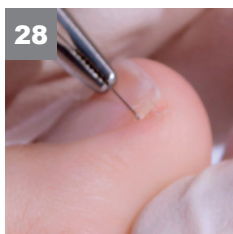

27 - 28 **TIP:**
If the brace should bend away when hooked in (e.g. with the strength „soft“), you can clamp the brace into the holding pliers directly behind the hook and then hook the brace under the nail. If you grip the brace far forward, it is more stable.




Once the brace is hooked in, fix it with your finger and release the pliers. Then grasp the pad with the COMBIped holding pliers and press it onto the nail.

29 - 30 **Check and optimize the tension of the brace:**
Before you glue the brace on, you must check the tension on the nail. Press the pad on the nail and discuss the tension with your patient.

The patient should not feel any unpleasant tension!

Only if the tension is too strong, or the nail has particularly strong curvatures or kinks, the brace will be slightly adjusted according to the nail curvature. Bend the brace slightly with your fingers or the combi bending pliers according to the shape of the nail. **It is better to select a weaker brace thickness from the outset than to bend the brace!**

For the brace to have its effect, the wire must always be less bent than the nail.

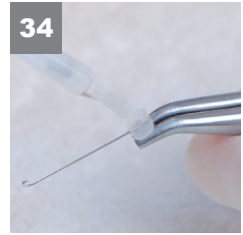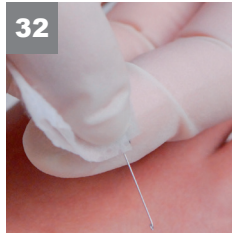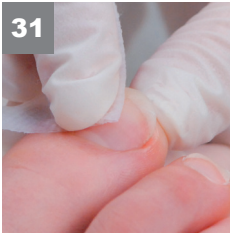31 - 32 **Clean** the wire hook, the adhesive surface of the brace pad and the nail with the alcohol swab. Let the nail and the brace **dry well** after cleaning, otherwise the glue will not hold.

Health risk: Risk of infection - see safety instructions

33 Open the **adhesive** and wipe the brush lightly from both sides of the bottle once at the edge.




34 Wet the adhesive surface of the pad **thinly** with glue.

Use the glue sparingly, if too much glue is used the brace will not hold on the nail. Keep the edge and the thread of the bottle clean. If necessary, clean with a swab, for example.



Health risk: Danger from sticking - see safety instructions

EN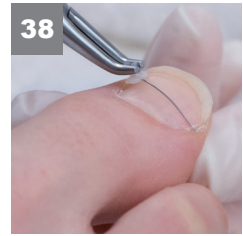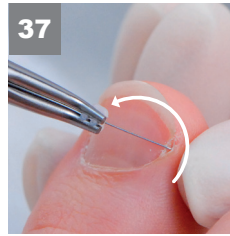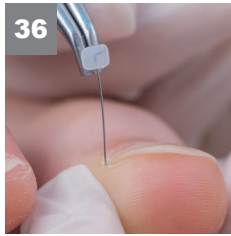

35 - 38 Hang the brace under the nail edge.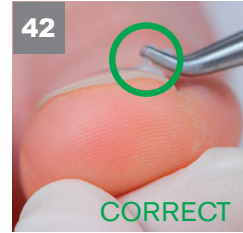
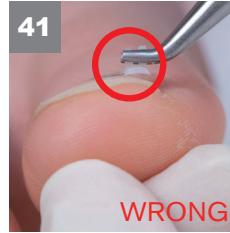


39 When hanging in, pull the brace only very slightly, otherwise the wire can tear the nail edge. The brace does not work by pulling but only by the spring force of the wire. **If the pulling force is too high, the wire will come off the adhesive pad.**

40 Press lightly with one finger for **about 1 minute** on the COMBIped holding pliers. Make sure that the COMBIped holding pliers are positioned correctly when pressing the pad. If the pliers point too steeply upwards, then the adhesive surface does not touch the nail and the brace cannot stick. Then loosen the COMBIped holding pliers and let the adhesive harden for **2 minutes**.

41 **WRONG**
The adhesive pad does not lie completely on the nail.

42 **CORRECT**
The adhesive pad lies completely on the nail.

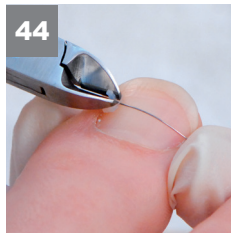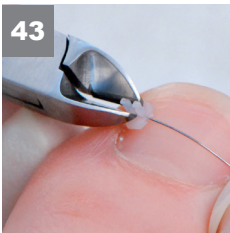
43 - 44 Use the side cutter to cut the holding pins as **short** as possible from the adhesive pad individually (each pin separately - 2 cuts).

45 - 46 Smooth the holding pad with a file.

Sealing (only if necessary) and aftercare:

Tampon the nail fold.

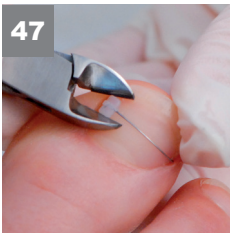EN

47

To **remove** the COMBIped brace, detach the pad from the nail with the side cutter and remove the wire carefully by unscrewing it from the nail fold.

Health risk: danger of stinging - see safety instructions

Remove the remains of the glue and smooth the nail.

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